

Sports Brace-Empowering People

## **PATELLA KNEE STRAP**

HCPCS: A4466 SIZE: Universal



## **FEATURES:**

- Strap functions dynamically as knee flexes and extends
- Easy to apply and adjust

**INDICATIONS:** Reduces pain for common joint conditions including Arthritis, Tendonitis and Chondromalacia. Ideal for minor sprains and strains of the knee ligaments.

**HOW TO APPLY / CARE INSTRUCTIONS:** Stand or sit with leg straight. Undo one side of strap. Place the front portion of the strap just below the kneecap at the natural indentation. Pull tight and fasten the hook and loop closure on the side. Loosen slightly if too tight. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat.

Visit us online at www.sportsbrace.net